



ATLAS URINE REAGENT STRIPS

1 PARAMETERS

Ketone in urine.

For *In-Vitro* and professional use only
Store at 15-30°C

WARNING AND PRECAUTIONS:

ATLAS Urine Reagent Strips are for in vitro diagnostic use and are intended for professional use. The "universal precautions" recommended by the Centers for Disease control should be adhered to whenever blood or body fluids are handled.

These precautions include wearing gloves. ATLAS Urine Reagent test strips may contain either diazonium salt or nitroferricyanide. Avoid contact with skin and mucous membranes; flush affected areas with copious amounts of water. Get immediate medical attention for eyes or if ingested. Exercise the normal precautions required for handling all laboratory reagents.

SUMMARY AND INTENDED USE:

ATLAS Urine reagent strip is a dip-and-read test strip and intended for use as an in vitro diagnostic and using urine specimens. The strip contains solid phase reagent areas affixed to a plastic support and is provided in a dry reagent format. This strip provides tests for, qualitatively and semi-quantitatively, **Ketone**, by the visual comparison with color charts of each concentration range. No additional reagents or laboratory equipment is required. The reagent strips are packaged in a plastic vial-containing desiccant. The test strips must be tightly capped in the plastic vial to assure reagent reactivity. The directions must

be followed exactly, and it is necessary to use fresh, well-mixed and uncentrifuged urine for optimal results.

CHEMICAL PRINCIPLES OF THE PROCEDURE:

Ketones: This test is based on the reaction of acetoacetic acid in the urine with nitroprusside. The resulting color ranges from tan when no reaction takes place to purple for positive reaction.

REAGENTS: (Based on dry weight at time of impregnation of 100strips)

Ketones	Sodium nitroprusside	20.0mg
	Magnesium sulfate	246.5mg

STORAGE:

Store at room temperature between 15°-30°(59°F-86°F). Do not store the strips in the refrigerator or freezer. Cap the bottle tightly.

Since the test strips are sensitive to specific environmental factors, such as moisture, heat and light, do not expose strips to these factors.

PROCEDURE FOR HANDLING THE STRIPS:

All unused strips must be stored in the original bottle. Do not remove desiccant from bottle. Transfer of the strips to another container may cause reagent strips to deteriorate and become unreactive. After taking out test strips, replace the cap, promptly and tightly. Do not touch test area of the strip. Do not use strips after expiration date. The work area should be clean and free of detergents and other contaminants.

SPECIMEN COLLECTION AND PREPARATION :

Use a clean, dry, unused vessel to collect the urine. Test the urine as soon as possible after collection. If testing cannot be done within an hour after voiding, refrigerate the specimen immediately and let it return to room temperature before testing.

PROCEDURE :

This procedure MUST BE FOLLOWED EXACTLY to achieve reliable test results.

1. Confirm that the product is within the expiration date shown on the label.
2. Remove the strip from the bottle and replace the cap immediately.
3. Inspect the strip. Discoloration or darkening of reagent areas may indicate deterioration. Do not use strip.

4. Dip the test strip completely for no more than 1 second in fresh, well-mixed, and uncentrifuged urine specimen. Excessive urine on the test strip may give rise to a wrong result. Remove the excessive urine by touching the plastic film on the rim of vessel. At this time, do not allow the reagent areas to touch the rim of vessel. Excessive urine may be removed by gently blotting the lengthwise edge on absorbent paper.
5. Compare the test results carefully with the color chart on the bottle label in good light. Proper reading time (30-60 seconds) is critical for optimal results. While comparing, keep the strip in a horizontal position to avoid possible interaction of chemicals by excessive urine. Changes in color that appears only along the edges of the test areas or after the correct timing period has passed are of no diagnostic significance.

QUALITY CONTROL:

The strips must be properly stored and handled before and during the testing. Reaction of reagent strips should be confirmed by testing known positive and negative specimens or multiple analyte controls containing normal and abnormal amounts of each of the analytes being tested.

RESULTS:

The results are obtained by direct comparison of test strip with the color blocks printed on the bottle label. No calculations or laboratory instruments are necessary.

LIMITATIONS OF PROCEDURES:

Substances the cause abnormal urine color, such as drugs containing azo dyes, nitrofurantoin and riboflavin may affect the readability of reagent areas on urinalysis reagent strips. The color development on the reagent pad may be masked, or a color reaction may be produced on the pad that could be interpreted visually and instrumentally as a false positive. It is therefore recommended that in case of doubt, the test should be repeated after withdrawal of the medication.

Ketones : Normal urine specimens ordinarily yield negative results with this reagent. False positive results may occur with highly pigmented urine specimens or those containing large amounts of levodopa metabolites.²

EXPECTED VALUSE :

Ketones : Ketone bodies should not be detected in normal urine specimens with this reagent. The concentration given : \pm (5mg/dl), +(15mg/dl), ++(40mg/dl), +++(80mg/dl) correlate well with the acetoacetic acid concentration in urine. The sensitivity of this test is 5mg acetoacetic acid per 100ml of urine. Detectable levels of ketone may occur with frequent vomiting, diarrhea, digestive disturbances, pregnancy, or severe physical exercise.

PERFORMANCE CHARACTERISTICS :

Specific performance characteristics of the ATLAS Urine reagent strips are based both on clinical and laboratory studies. A study done at two clinical sites involving 94 patient samples compared ATLAS Urine reagent strips to a competitor's strip. 100% agreement within one color block was obtained for all analytes except protein. Protein gave a greater than 95% agreement. The lower agreement may be reflective of the technician's interpretation of the negative versus trace color block with both the Urine Reagent strips and the competitive strip. Parameters of importance to the user are sensitivity, limits of test, specificity, accuracy, precision and stability. Sensitivity and limits of tests are the generally detectable levels of each test described previously. The sensitivity depends upon several factors; the variability of color perception; the presence or absence of inhibitory factors typically found in urine, the specific gravity, ascorbic acid and pH; and lighting conditions when the product is read visually. The tests have been developed to be specific for the constituent to be measured with the exception of interferences listed previously. Exact agreement between visual results and instrumental results might not be found because of the inherent differences between the perception of the human eye and the optical system of the instruments. It is for this reason that each user is encouraged to develop his own standards for performance. The stability test has been developed by statistical procedure for various environmental conditions.

Ketones : The reaction of this reagent pad is caused by acetoacetic acid in urine, acetone or beta-hydroxybutyric acid makes no significant contribution to this test. Ordinarily, the reagent area detects 5.0mg of acetoacetic acid in 100ml urine. Some high specific gravity and low pH urines may give reactions up to and including trace level (5.0mg/dl).

BIBLIOGRAPHY :

- 1.NCCLS. Protection of Laboratory Workers from infectious Disease Transmitted by Blood and Tissue. NCCLS Doc. M29-P79, May, 342-347(1985)
- 2.Kaplan L.A. and Pesce A.J., Clinical Chemistry Theory, Analysis and Correlation, C.V.Mosby,pp. 1004-1007, (1984.)
- 3.Henry, J.B.et, al Clinical Diagnosis and Management by Laboratory methods, 16th ed. Philadelphia Saunders, 579-608(1979).
- 4.Paterson. P.et.al., Saunders, 579-608(1979).

GENERAL REFERENCES:

- 1.Free, A H., and Free, H. M., Urinalysis Clinical Discipline of Clinical Science, CRC Crit. Rev. Clin. Lab. Sci., 3(4), 481-531(1972).
- 2.Graff. L., Handbook of Routine Urinalysis, Philadelphia, J. B. Lippincott Co.,(1983).
- 3.Jungreis, E., Spot Test Analysis, N.Y., John Wiley & Sons(1985)
- 4.Kart, R. M., et. al., A Primer of Urinalysis, 2nd ed., N. Y., Harper and Row(1963)
- 5.Scheer, W. D., Am J. Clin. Pathol., 87, 86-93 (1987)

ATLAS Medical

William James House,

Cowley Road, Cambridge, CB4 0WX, UK

Tel: ++44 (0) 1223 858 910

Fax: ++44 (0) 1223 858 524

PPI053A01

Revision B (01.12.2004)