


LDH

For *in-vitro* diagnostic use only.

Store at 2-8°C.

PRINCIPLE OF THE METHOD

Lactate dehydrogenase (LDH) catalyses the reduction of pyruvate by NADH, according to the following reaction:

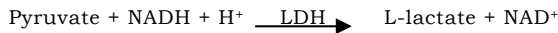

The rate of decrease in concentration of NADPH, measured photometrically, is proportional to the catalytic concentration of LDH present in the sample.

CLINICAL SIGNIFICANCE

Lactate dehydrogenase (LDH) is an enzyme with wide tissue distribution in the body.

The higher concentrations of LDH are found in liver, heart, kidney, skeletal muscle and erythrocytes.

Increased levels of the enzyme are found in liver disease, myocardial infarction, renal disease, muscular dystrophy and anemia' 4,5.

Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

REAGENTS

R 1 Buffer	Phosphate pH 7.8 Pyruvate	50 mmol/L 0.6 mmol/L
R 2 Substrate	NADH	0.18 mmol/L

PREPARATION

Working reagent (WR):

Dissolve 1 tablet of R 2 in 3 mL of R 1.

Cap and mix gently to dissolve contents.

Stability: 2 days at 2-8°C or 12 hours at room temperature (15-25°C).

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use.

Do not use the tablets if appears broken.

Do not use reagents over the expiration date.

Signs of reagent deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 340 nm < 1.00.

ADDITIONAL EQUIPMENT

- Spectrophotometer or colorimeter measuring at 340 nm. -
- Thermostatic bath at 25°C, 30°C or 37°C (± 0.1°C) - Matched cuvettes 1.0 cm light path.
- General laboratory equipment.

SAMPLES

Serum'. Separated from cells as rapidly as possible. Do not use oxalates as anticoagulants since they inhibit the enzyme.

Do not use haemolysed samples.

Stability: 2 days at 2-8°C.

PROCEDURE

1. Assay conditions:

Wavelength: 340 nm
Cuvette : 1 cm light path
Constant temperature 25°C / 30°C / 37°C

2. Adjust the instrument to zero with distilled water or air.

3. Pipette into a cuvette:

	25° - 30°C	37°C
WR (mL)	3.0	3.0
Sample (µL)	100	50

4. Mix, incubate for 1 minute.

5. Read initial absorbance (A) of the sample, start the stopwatch and read absorbances at 1 min intervals thereafter for 3 min.

6. Calculate the difference between absorbances and the average absorbance differences per minute (ΔA/min).

CALCULATIONS

$$25^{\circ} - 30^{\circ}\text{C } \Delta\text{A}/\text{min} \times 4925 = \text{U/L LDH}$$

$$37^{\circ}\text{C } \Delta\text{A}/\text{min} \times 9690 = \text{U/L LDH}$$

Units: One international unit (IU) is the amount of enzyme that transforms 1 µmol of substrate per minute, in standard conditions. The concentration is expressed in units per litre of sample (U/L).

Temperature conversion factors

To correct results to other temperatures multiply by:

Assay temperature	Conversion factor to		
	25°C	30°C	37°C
25°C	1.00	1.33	1.92
30°C	0.75	1.00	1.43
37°C	0.52	0.70	1.00

QUALITY CONTROL

Control sera are recommended to monitor the performance of assay procedures.

If control values are found outside the defined range, check the instrument, reagents and technique for problems.

Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

REFERENCE VALUES

25°C 30°C 37°C
120-240 U/L 160-320 U/L 230-460 U/L

These values are for orientation purpose; each laboratory should establish its own reference range.

PERFORMANCE CHARACTERISTICS

Measuring range: From detection limit of 5.5 U/L to linearity limit of 1453 U/L. If the results obtained were greater than linearity limit, dilute the sample 1/10 with NaCl 9 g/L and multiply the result by 10.

Precision:

	Intra-assay (n=20)		Inter-assay (n=20)	
	Mean (U/L)	SD	CV%	
Mean (U/L)	338	548	345	553
SD	6.43	5.11	3.81	8.90
CV%	1.90	0.93	1.10	1.60

Sensitivity: 1 U/L = 0,000947 ΔA/min.

Accuracy: Results obtained using Atlas reagents (y) did not show systematic differences when compared with other commercial reagents (x). The results of the performance characteristics depend on the analyzer used.

INTERFERENCES

Haemolysis interferes with the assay.

Some anticoagulants such as oxalates interfere with the reaction.

A list of drugs and other interfering substances with LDH determination has been reported by Young et. al".

BIBLIOGRAPHY

1. Pesce A. Lactate dehydrogenase. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1124-117, 438.
2. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
3. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
4. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
5. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

ATLAS MEDICAL

Unit 4, William James House

Cowley Rd, Cambridge, CB4 0WX

Tel: ++44 (0) 1223 858 910

Fax: ++44 (0) 1223 858 524

PPI204A01

Revision A (05.07.2005)