


Urea Berthelot

**Enzymatic –colorimetric test
."Berthelot"**

For in vitro diagnostic use
only.
Store at 2-8 °C

PRINCIPLE OF THE METHOD

Urea in the sample is hydrolized enzymatically into ammonia (NH₄) and carbon dioxide (CO₂). Ammonia ions formed reacts with salicylate and hypochlorite (NaClO), in presence of the catalyst nitroprusside, to form a green indophenol:

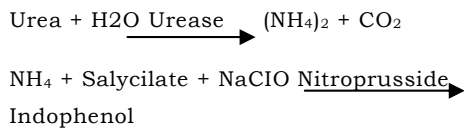

The intensity of the color formed is proportional to the urea concentration in the sample.

CLINICAL SIGNIFICANCE

Urea is the final result of the metabolism of proteins; it is formed in the liver from its destruction. Elevated urea can appear in blood (uremia) in: diets with excess of proteins, renal diseases, heart failure, gastrointestinal hemorrhage, dehydration or renal obstruction. Clinical diagnosis should not be made on a single test result; it should integrate clinical and other laboratory data.

REAGENTS

R 1 Buffer	Phosphate pH 6.7 EDTA Sodium salicylate Sodium nitroprusside	50 mmol/L 2 mmol/L 60 mmol/L 3.2 mmol/L
R 2 NaClO	Sodium hypochlorite (NaClO) Sodium hydroxide	140 mmol/L 150 mmol/L
R 3 Enzymes	Urease	30000 U/L
UREA CAL	Urea aqueous primary standard 50 mg/dL	

PRECAUTIONS

R2: Irritant (Xi): Irritating to eyes and skin. S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S37/39: Wear suitable gloves and eye/face protection.

PREPARATION

- Working reagent (WR): Dissolve (→) the powder content of R 3 Enzymes in one bottle of R 1 Buffer. Cap and mix gently to dissolve contents.
Stability: 4 weeks in the refrigerator (2-8°C) or 7 days at room temperature (15-25°C).
- R 2 NaClO is ready to use.

STORAGE AND STABILITY

All the components of the kit are stable until the expiration date on the label when stored tightly closed at 2-8°C, protected from light and contaminations prevented during their use. Do not use reagents over the expiration date.

UREA CAL

Once open is stable up to 1 month when stored

tightly closed at 2-8°C, protected from light and contaminations prevented during their use.

Signs of reagent deterioration:

- Presence of particles and turbidity.
- Blank absorbance (A) at 340 nm > 0.32.

ADDITIONAL EQUIPMENT

- Spectrophotometer or colorimeter measuring at 580 nm.
- Matched cuvettes 1.0 cm light path.
- General laboratory equipment.

SAMPLES

- Serum or heparinized plasma: Do not use ammonium salts or fluoride as anticoagulants.
 - Urine: Dilute sample 1/50 in distilled water. Mix. Multiply results by 50 (dilution factor). Preserve urine samples at pH < 4.
- Urea is stable at 2-8°C for 5 days;

PROCEDURE

1. Assay conditions:
Wavelength: 580 nm
Cuvette : 1 cm light path
Temperature 37°C / 15-25°C
2. Adjust the instrument to zero with distilled water.
3. Pipette into a cuvette:

	Blank	Standard	Sample
WR (mL)	1.0	1.0	1.0
Standard ^{10⁻⁶ 2⁻¹} (µL)	--	10	--
Sample (µL)	--	--	10

4. Mix and incubate 5 min at 37°C or 10 min at room temperature (15-25°C).
5. Pipette:

	Blank	Standard	Sample
R2 (mL)	1.0	1.0	1.0

6. Mix and incubate 5 min at 37°C or 10 min at room temperature (15-25°C).
7. Read the absorbance (A) of the samples and calibrator, against the Blank. The colour is stable for at least 30 minutes at 15-25°C.

CALCULATIONS

$(A)_{\text{Sample}} \times 50 (\text{Standard conc.}) = \text{mg/dL urea in the } (A)_{\text{Standard}}$
sample

10 mg/L urea BUN divided by 0.466 = 21 mg/L urea = 0.36 mmol/L urea.

Conversion factor: mg/dL x 0.1665 = mmol/L.

QUALITY CONTROL

Control sera are recommended to monitor the performance of assay procedures. If control values are found outside the defined range, check the instrument, reagents and calibrator for problems. Each laboratory should establish its own Quality Control scheme and corrective actions if controls do not meet the acceptable tolerances.

REFERENCE VALUES

Serum : 15- 45 mg/dL (2.49-7.49 mmol/L)
Urine : 20 - 35 gr/24 h.
These values are for orientation purpose; each laboratory should establish its own reference range.

PERFORMANCE CHARACTERISTICS

Measuring range: From *detection limit* of 0.3 mg/dL to *linearity limit* of 200 mg/dL. If the results obtained were greater than linearity limit, dilute the sample 1/2 with NaCl 9 g/L and multiply the result by 2.

Precision:

	Intra-assay (n=20)	
Mean (mg/dL)	40.0	139
SD	1.27	3.50
CV (%)	3.17	2.50

Inter-assay (n=20)

40.0 142

1.86 3.75

4.64 2.63

Sensitivity: 1 mg/dL = 0.00505 A.

Accuracy. Results obtained using ATLAS reagents (y) did not show systematic differences when compared with other commercial reagents (x). The results obtained using 50 samples were the following:

Correlation coefficient (r): 0.9941.

Regression equation: $y = 0.9972x + 0.011$.

The results of the performance characteristics depend on the analyzer used.

INTERFERENCES

It is recommended to use heparin as anticoagulant. Do not use ammonium salts or fluoride.

A list of drugs and other interfering substances with urea determination has been reported by Young et. al^{4,5}

NOTES

1. Glassware and distilled water must be free of ammonia and ammonium salts
2. Calibration with the aqueous standard may cause a systematic error in automatic procedures. In these cases, it is recommended to use a serum Calibrator.
3. Use clean disposable pipette tips for its dispensation.

BIBLIOGRAPHY

1. Kaplan A. Urea. Kaplan A et al. Clin Chem The C.V. Mosby Co. St Louis. Toronto. Princeton 1984; 1257-1260 and 437 and 418.
2. Tabacco A et al. Cin Chem 1979; 25: 336-337.
3. Fawcett J K et al. J Clin Path 1960; 13: 156-169.
4. Young DS. Effects of drugs on Clinical Lab. Tests, 4th ed AACC Press, 1995.
5. Young DS. Effects of disease on Clinical Lab. Tests, 4th ed AACC 2001.
6. Burtis A et al. Tietz Textbook of Clinical Chemistry, 3rd ed AACC 1999.
7. Tietz N W et al. Clinical Guide to Laboratory Tests, 3rd ed AACC 1995.

ATLAS Medical
William James House,
Cowley Road, Cambridge, CB4 0WX, UK
Tel: ++44 (0) 1223 858 910
Fax: ++44 (0) 1223 858 524

PPI030A01
Revision B (02.07.2005)